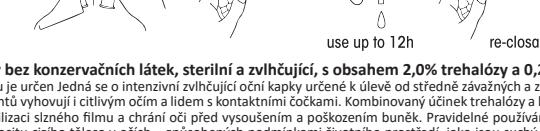
PILOSET BIODROP PLUS

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English • Preservative-free, sterile, moisturizing and lubricating eye drop with 2.0% trehalose and 0.25% hyaluron

What it is and what it is intended to This is an intensive moisturizing eye drop intended to relieve moderate and severe dry eye symptoms. The preservative free eye drops suit also sensitive eyes and contact lens users. The combined action of trehalose and hyaluronan provide hydration, lubrication and stabilization of the tear film, protecting the eye from drying and cell damage. Regular use relieves dry eye symptoms — itching, burning and foreign body sensation in the eyes — caused by environmental factors such as dry air, wind, smoke, air pollution, excessive light, visual display work or ocular surgery and contact lens use. The intensive moisturizing eye drops suit long-term and frequent use. This is a medical device developed and manufactured in Finland.

InSTRUCTIONS FOR USE Apply 1–2 drops in the eyes 1–3 times daily, as needed. Open the aluminium bag, take off one vial from the strip and open it by turning and pulling the cap. Put one drop in the eye by squeezing gently the body of the vial. The vial can be reclosed (see pictures) and the remaining product used up in 12 hours. You can use moisturizing drops during contact lens wear.

Precautions and warnings • Do not use if you are sensitive to any of the ingredients. • Do not use if the vial is damaged. • Each vial should be used by one user only. • Do not touch the eye or contact lens with the tip of the vial. Do not touch the tip of the vial with fingers. • Do not use after the expiry date. • Mild temporary blurring of vision may be experienced immediately after application. It is therefore advisable to wait a few seconds before continuing any activity that requires clear vision. • Leave at least 10–15 minutes between applications of any other ophthalmic preparation.

• Keep out of the reach and sight of children. • Store at 15–30 °C, protect from direct sunshine and freezing. • Discard the empty vial appropriately.

Composition 2.0% trehalose, 0.25% hyaluronan in sterile isotonic, low-phosphate-buffered solution. No materials of animal origin.

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Oy Finnsup Ab Pääskykalliointie 5 FI-21420 Lieto, Finland www.piloset.fi